**DE**

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## Setup

1. Insert batteries: See pages 1 to 5.

The device vibrates twice to confirm its function.

2. Connect the plug with the AUX-socket of a *lisa-* or *signolux*-receiver (page 6).

## Status indication „Low battery“

Depleted batteries are indicated by the following signs:

- The status-LED „BAT“ blinks every 4 seconds
- When activating the pad, vibration is as follows:

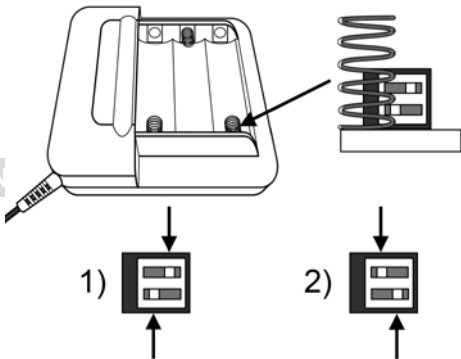
Please replace the batteries as soon as possible!

## **Using rechargeable batteries**

The vibrating pad can also be powered by rechargeable batteries of size AA.

The different voltages of normal batteries and rechargeable ones make the following adjustment necessary:

1. Open the battery compartment
2. Remove the batteries
3. Underneath the front contact spring is an opening with a small adjuster switch, as depicted on the next page:

Position 1): When using regular batteries

Position 2): When using rechargeable batteries

With the help of a pointed object, change the position of the small switches accordingly.

Please note: Depleted rechargeable batteries need to be recharged in an external battery charger.

Should regular batteries be used again, please re-set the adjustment switch accordingly.

EN

Designed in Europe

Made in China

**This device complies with the following EC directives:**

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 30 / EC EMC-directive
- 2014 / 35 / EC LVD-directive

The complete declaration of conformity is available at  
[www.humantechnik.com](http://www.humantechnik.com)

This product must not be handled as ordinary household waste!  
Please dispose of used batteries according to your local regulations!

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**Humantechnik GHL AG**
**Sarabec Ltd.**