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Spectacle frames labelling ENGLISH

Field of use

Safilo spectacle frames are class I medical devices (Rule 1 Chapter III of Annex VIII of (EU) Regulation 2017/745), which are principally intended for use in connection with ophthalmic prescription lenses for correcting visual problems. Spectacle frames comply with the requirements set out in (EU) Regulation 2017/745 and meet the harmonized standard EN ISO 12870.

Warning

Adverse contact reactions to materials and treatments used for the frames in particularly sensitive or allergic subjects cannot be ruled out. The frame comes with clear lenses that are not intended for use but are designed to only prevent frame warping and allow for a more realistic try-on.

Storage / Cleaning

Store the spectacle frames in their case at a temperature between -10°C and +35°C. Avoid exposure to the sun in closed, non-air-conditioned areas (e.g. car dashboard), because high temperatures may alter the features of the frames. In case of contact with agents such as salt water (seawater), swimming pool water, chemical products such as make-up, hairspray, sun lotions, and insect repellents, we recommend that you immediately remove such agents using a soft damp cloth. In case of wear and tear (e.g. scratched, opaque), we recommend the use of original replacement spare parts. Use a damp cloth and mild soap to clean the frames, then dry them with a soft, clean cloth. Do not use solvents (e.g. alcohol, acetone) or aggressive detergents which may alter the features of the frames.

Marking

The model identification mark and the date of manufacture are on the inside of the product and/or on the individual packaging. Marking in accordance with EN ISO 12870.

Reporting/Contact

In case of complaints or anomalies, should you not be able to contact the store where you purchased the product, please forward your request to: info@safilo.com
READ and KEEP

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